

Medical Book Card



POCKET PDR[®]

User's Guide

Quick Reference Guide

To Use the Menus

1. Press MENU and highlight the Drugs menu or the Class menu.
2. Highlight an entry by typing its name or pressing \uparrow or \downarrow .
3. Press ENTER.
4. Select items from the submenus, if needed, by highlighting them and pressing ENTER.
5. Use \downarrow , \odot , or SPACE to scroll.

To Search for Words

1. Press SEARCH.
2. Type one or more words.
3. Press ENTER.
4. Hold \star and press \odot to see the locations of matches.
5. Press ENTER to see a match or MENU to see the matches listed on the menus.
6. To expand a search, press MORE.

To Use Notes

1. When the text of the book is on screen, press LIST.
2. Press ENTER to select *Add note*.
3. Press ENTER or type your own note title and then press ENTER.
4. To find a note in the text, press LIST, highlight a note title, and then press ENTER.

Quick Keys*

A	Goes to <u>A</u> dverse Reactions topic.
C	Goes to <u>C</u> ontraindications topic.
D	Goes to <u>D</u> osage and Administration topic.
E	Goes to <u>E</u> quivalents menu.
G	Goes to <u>G</u> eneric Name topic.
H or S	Goes to <u>H</u> ow <u>S</u> upplied topic.
I	Goes to <u>I</u> ndications and Usage topic.
M	Goes to <u>M</u> anufacturer topic.
N	Goes to <u>P</u> regnancy and <u>N</u> ursing topic.
P	Goes to <u>P</u> harmacologic Class topic.
T	Goes to <u>T</u> herapeutics Class index.
W	Goes to <u>W</u> arnings topic.
X	Goes to <u>D</u> rug Interactions topic.

* You can use the Quick Keys only when the text of a drug monograph is on the screen.

License Agreement

READ THIS LICENSE AGREEMENT BEFORE USING THE POCKET PDR®. YOUR USE OF THE POCKET PDR DEEMS THAT YOU ACCEPT THE TERMS OF THIS LICENSE. IF YOU DO NOT AGREE WITH THESE TERMS, YOU MAY RETURN THIS PACKAGE WITH PURCHASE RECEIPT TO THE DEALER FROM WHICH YOU PURCHASED THE POCKET PDR AND YOUR PURCHASE PRICE WILL BE REFUNDED. The POCKET PDR means the software product, hardware, and documentation found in this package and FRANKLIN means Franklin Electronic Publishers, Inc.

LIMITED USE LICENSE

All rights in the POCKET PDR remain the property of FRANKLIN. Through your purchase, FRANKLIN grants you a personal and nonexclusive license to use the POCKET PDR. You may not make any copies of the POCKET PDR or of the preprogrammed data stored therein, whether in electronic or print format. Such copying would be in violation of applicable copyright laws. Further, you may not modify, adapt, disassemble, decompile, translate, create derivative works of, or in any way reverse engineer the POCKET PDR. You may not export or re-export, directly or indirectly, the POCKET PDR without compliance with appropriate governmental regulations. The POCKET PDR contains Franklin's confidential and proprietary information which you agree to take adequate steps to protect from unauthorized disclosure or use. This license is effective until terminated. This license terminates immediately without notice from FRANKLIN if you fail to comply with any provision of this license.

Contents

License Agreement.....	1
Publishers' Note	3
Key Guide.....	4
Installing Book Cards.....	6
Selecting a Book.....	6
Changing the Settings	7
Viewing a Demonstration.....	7
Finding Drug Information	8
Finding Therapeutic Classes....	9
Searching for Words	10
Highlighting Search Words	12
Expanding Your Searches	12
Using Notes.....	13
Looking Up Words in Other Books.....	14
Resetting the Medical Book System.....	15
Book Card Care	15
Specifications and Patents.....	16
FCC Notice	16
Limited Warranty (U.S. only)	17
Index	18

► About Book Card Compatibility

IMPORTANT: This book card is compatible with the Medical Book System™ platform, and with the 1450 and 1870 BOOKMAN® platforms **only**. This book card will not function in any other Franklin BOOKMAN platform.

The Medical Book System platform can use Franklin BOOKMAN book cards as well as Medical Book System book cards. However, except for this book card, Franklin BOOKMAN platforms cannot use Medical Book System book cards.

► For More Information

To learn more about the Medical Book System or other products from Franklin Electronic Publishers, call 800-266-5626 or visit the Franklin Web site at www.franklin.com.

Publishers' Note

Entries in this *Pocket PDR*[®] are drawn from FDA-approved product labeling as published in *Physicians' Desk Reference*[®] or supplied by the manufacturer. The entries have been compiled by a staff of experienced pharmacists under the guidance of PDR's distinguished Board of Advisors. Every effort has been made to assure the accuracy of each entry, but it is essential to bear in mind that the information presented here is merely a synopsis of key points in the official labeling, and that the complete labeling contains additional precautionary information that may be of significance in a particular case. Similarly, please remember that only common and dangerous adverse reactions and interactions are included here, and that numerous less prevalent adverse effects may be reported in the underlying labeling. If an entry leaves any question unanswered, be sure to seek additional information from *Physicians' Desk Reference*, *PDR*[®] *Supplements*, *PDR for Ophthalmic Medicines*[™] or the manufacturer.

Physicians' Desk Reference assumes no obligation to obtain and include any information in these entries other than that

provided to it by the manufacturer. Neither the publisher, Thomson Healthcare Inc., nor the producer of this unit, Franklin Electronic Publishers, Inc., warrant, guarantee, or advocate the use of any product described herein, nor do they warrant that the information is accurate or that the unit will perform without interruption or free of errors. The publisher and producer do not assume, and expressly disclaim, any liability for error or omissions in the information contained herein and for misuse of any of the products listed, nor shall they be liable for any claim arising out of or relating to use of *Pocket PDR*. For additional information on limitations of liability and warranty disclaimers, consult this User's Guide. All use of this product is subject to the terms and conditions of the License Agreement. Information in this book card is current through the 2004 editions of *PDR* and *PDR for Ophthalmic Medicines*. For later revisions in prescribing information, always check *PDR* Supplements or the latest printed editions. Most drugs listed in *Pocket PDR* are available only by prescription. Patients should consult a physician before using such medications.

Key Guide

MORE

SPEC

SEARCH

LIST

BACK

CAP

CARD

CLEAR

ENTER

HELP

MENU

ON/OFF

SPACE

?*

Color Keys

(red) Expands a word search.

(green) Displays the Outline location of the current text.

(yellow) Displays the word search screen.

(blue) Lets you add, find, or remove a note.

Function Keys

Erases typed letters, or backs up to the previous screen.

Shifts keys to type capitals or punctuation.

Exits the currently selected book.

Clears an entry or search and goes to the default state.

Enters a word, selects a menu item, or starts the highlight in text.

Displays help messages.

Displays the main menus.

Turns the platform on or off.

At entry screens, types a space. At menus and text, pages down.

At menus, displays the title of a highlighted item. With **CAP**, types an asterisk (*) to stand for letters in a word. At text, displays the current Outline location of the text.

Function Key Combinations*

 + **BACK**

Goes to the top level of a menu from a submenu.

 + **CARD**

Transfers a highlighted word between installed book cards.

 + **ENTER**

At a highlighted menu item, displays the related text. At text, highlights special markers (e.g., footnotes, cross-references, tables, etc.).

 + **Q-P**

Types numbers.

Direction Keys

Pages up or down.



Moves the cursor, text, or highlight.

Direction Key Combinations*

 +  **(NEXT)**

At text, displays the next paragraph, table row, or search match.

 +  **(PREV)**

At text, displays the previous paragraph, table row, or search match.

* Hold the first key while pressing the second.

Key Guide


- CAP + ↑** (HOME) Goes to the top of a menu or list. At text, goes to the previous Outline title.
- CAP + ↓** (END) Goes to the bottom of a menu or list. At text, goes to the next Outline title.

Quick Keys*

- A** Goes to Adverse Reactions topic.
- C** Goes to Contraindications topic.
- D** Goes to Dosage and Admistration topic.
- E** Goes to Equivalents menu.
- G** Goes to Generic Name topic.
- H or S** Goes to How Supplied topic.
- I** Goes to Indications and Usage topic.
- M** Goes to Manufacturer topic.
- N** Goes to Pregnancy and Nursing topic.
- P** Goes to Pharmacologic Class topic.
- X** Goes to Drug Interactions topic.
- T** Goes to Therapeutics Class index.
- W** Goes to Warnings topic.

* You can use the Quick Keys only when the text of a drug monograph is on the screen.

► Understanding the Keys

This User's Guide describes the function of keys using this book card in the Medical Book System™. When this book card is used in the 1450 and 1870 BOOKMAN® platforms, some functions are performed using keys different from those indicated in this User's Guide. For example, use **FN** in places where you are instructed to use . Read the User's Guide and Tutorial for your platform to see the proper keys required to perform a **NEXT** or **PREV** or other direction key combinations.

In addition, the functions of the keys may vary according to which book card is installed and selected in the platform. To learn how to use a particular book card, read its User's Guide.

Note: Keys referred to as “color” keys in this User's Guide are referred to as “touch” keys in the 1450 BOOKMAN platforms and “function” keys in the 1870 BOOKMAN platforms.


► Understanding the Quick Keys

When a drug monograph is on the screen, you can use the quick keys to quickly view its different parts, its equivalents, or the Therapeutics Class index.

Installing Book Cards

CAUTION: Never install or remove a book card when the platform is turned on. If you do, information entered in any installed book cards may be erased.

1. Turn the platform off.
2. Turn the platform over.
3. Remove the blank protector or any other book card that might be installed.
4. Align the book card tabs with the notches in a card slot.

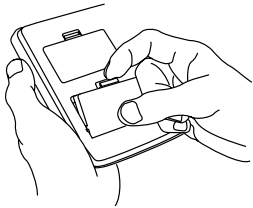

5. Press the book card until it snaps into the slot.

▶ Removing Book Cards

CAUTION: When you remove a book card from a platform, information entered in that book card may be erased.

Selecting a Book

If you have installed two book cards in the platform, you can select which book you want to read.

1. Turn the platform on.
2. Press CARD.



3. Use ← or → to highlight the book you want to use.



4. Press ENTER to select it.


▶ About Screen Illustrations



Some screen illustrations in this User's Guide may differ slightly from what you see on screen. This does not mean that your product is malfunctioning.


Changing the Settings



Using the Setup menu, you can adjust the type size, default topic, default state, shutoff time, and screen contrast of this book.

The type size sets how large the characters appear on screen. The shutoff time sets how long your Medical Book System stays on if you forget to turn it off.

The default topic sets which topic will appear first when you hold  and press **ENTER** to view a drug monograph. The default state sets the screen that appears when you press **CLEAR**.

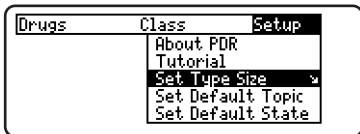
1. Press **MENU**.
2. Use  or  to highlight *Setup*.
3. Highlight the setting you want and press **ENTER**.



4. Use  or  to change the setting. Or press **BACK** to leave the setting unchanged.
5. Press **ENTER** to select it.

Viewing a Demonstration

Before you start using this book, you may want to see a brief demonstration. Press **MENU**, highlight the Setup menu and then use the arrow keys to highlight *View Demo*.

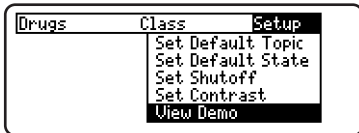

Press **ENTER** to select it. To stop the demonstration, press **CLEAR**.

► Help is Always at Hand

You can view a help message at virtually any screen by pressing **HELP**. To exit help, press **BACK**.

To read a tutorial about this book, select *Tutorial* from the Setup menu. For more information about the Pocket PDR[®], select *About PDR* from the Setup menu.


► Follow the Arrows

The flashing arrows at the right of the screen show which arrow keys you can press to move through menus or view more text.

Finding Drug Information

The simplest way to find drug information is to type a brand or generic drug name at the Drugs menu. Brand names are capitalized on the Drugs menu, but you do not need to type capitals.

1. Press **CLEAR**.
2. If needed, press **MENU** and highlight the **Drugs** menu.
3. Type a drug name. For example, type *mar* to highlight *Marinol*.

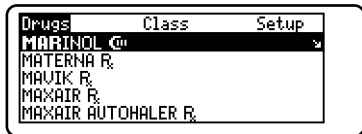

To delete a letter, press **BACK**.

You can also highlight a menu item by using \downarrow or \uparrow .


The slanted arrow indicates that the menu item has a submenu.

4. When the drug is highlighted, press **ENTER** to select it.

If you entered a generic name, select one of the brand name drugs that contain the selected generic name.

If you entered a brand name, the monograph's topics appear.

Note: You can go directly from the Drugs menu to the default topic of the monograph by holding \star and pressing **ENTER**.

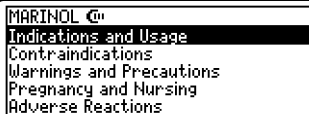

5. Highlight a topic and press **ENTER** to view the drug monograph.

Indications and Usage
Treatment of anorexia associated with weight loss in AIDS patients and nausea and vomiting associated with chemotherapy when conventional treatment has failed.

Contraindications

6. To go quickly to another topic, use the **Quick Keys**.


To learn more about the Quick Keys, read the "Key Guide."

7. Use \downarrow , DN , or **SPACE** to scroll.
8. To go to the next or previous topic, hold **CAP** and use \downarrow or \uparrow .
9. Press **CLEAR** when finished.

Finding Drug Information

▶ If You Misspell a Drug Name

When you type letters that do not match a drug on the Drugs menu, the spelling correction entry screen appears. You can press **BACK** repeatedly to exit it. Or you can enter the misspelled drug name to view corrections.

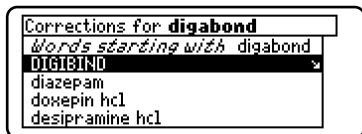

You can select *Words starting with...* to view completions, if any, or select a correction to view its monograph.


▶ Finding Drug Equivalents and Therapeutic Classes

When reading a drug monograph, you can quickly find its equivalents or a list of drugs in its therapeutic class. To find a drug's equivalents, if any, press **E** and then select a monograph. To find a list of drugs in its therapeutic class, press **T**.

Finding Therapeutic Classes

Every drug in the Pocket PDR® is classified by its therapeutic use.

1. Press **CLEAR**.
2. If needed, press **MENU** and highlight the **Class** menu.
3. Start typing the name of a therapeutic class. For example, type **M** for *Muscular/Skeletal System*.

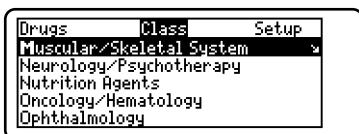

To de-select a letter, use **BACK**. You can also use **↓** or **↑** to highlight a class.

4. When the class is highlighted, press **ENTER**.
Enter through the sub-menus until you see a list of drugs.
5. Highlight a drug and press **ENTER**.
For example, select *Evista*.
6. Highlight a topic and press **ENTER**.

Drug Interactions


Avoid concomitant use with anion exchange resins; cholestyramine decreases absorption. Monitor PT/INR with warfarin and other anticoagulants. Caution with other highly protein-bound drugs (eg, diazepam, diazoxide, lidocaine).

Finding Therapeutic Classes

7. To go quickly to another topic, use the **Quick Keys**.
To learn more about the Quick Keys, read the “Key Guide.”
8. Use **↓**, **(DN)**, or **SPACE** to scroll.
9. To go to the next or previous topic, hold **CAP** and use **↓** or **↑**.
10. Press **CLEAR** when finished.

► Where in the Book Am I?

When you are reading the text of a drug monograph or table, you can see the Outline location of the current text by pressing **SPEC** or **?***.

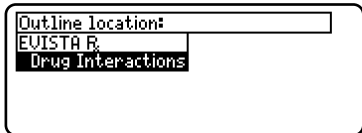

To scroll up to a higher level of the Outline, use **↑**.

To display the full title of an item, press **?***. This is especially useful when the title ends in ellipses.


Press **BACK** to go back to the text that you were reading. Or press **ENTER** to go to the text of the highlighted item.

Searching for Words

Word searches find all the occurrences or matches of a word or word phrase in this book.

You can type up to 99 characters in a single search, but you cannot search for short, common words such as *the* and *is*.


1. Press **SEARCH**.
2. Type your search words. For example, type *flu*.

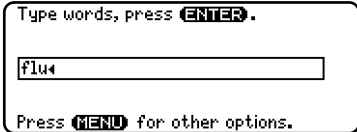


To delete a letter, use **BACK**.

You do not need to type capitals or punctuation to find matches.

3. Press **ENTER** to search.

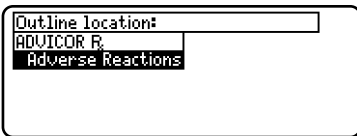

Searching for Words

This is the Outline location of the first match of your search words. To see the Outline location of the next or previous matches, if any, hold  and press **DN** or **UP**.

4. Press **ENTER** to see the match.

Flushing, asthenia, **PIU** syndrome, headache, infection, pain, GI effects, hyperglycemia, pruritis, rash.

Dosage and Administration
Routes: 218 *hrs:* Initial: 500mg-
20mg qhs. *Titrates:* Increase by no

Your search words are boxed.

To go quickly to another topic, use the Quick Keys. To learn more about the Quick Keys, read the “Key Guide.”

5. Press **MENU** to see the matches listed on the menus.

After a word search, only matching entries are listed on the menus.

6. Use the arrow keys to highlight a menu item and then press **ENTER**. If needed, select an item from a submenu.


7. Press **SEARCH** or **CLEAR** to clear your search.

► If You Misspell a Search Word

If you enter a misspelled search word, a list of corrections appears.

Corrections for **psoriasis**

psoriasis
sources
psoriatic
seriously
Sumycin

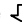Use  to highlight a correction and then press **ENTER**.


► Searching for Parts of Words

If you want to find words that use prefixes, suffixes, or other parts of words, type an asterisk (*) in place of the missing letters. For example, type *bacter** and then press **ENTER**.

To type an asterisk, hold **CAP** and press **?***.

Matches for **bacter***

Bacteroides
bacteremia
bacteremias
bacteria
bacterial

Use  to highlight a match and then press **ENTER**.

Highlighting Search Words

You can also search for a word by highlighting it in text.

1. At the text, press **ENTER**.

Dosage and Administration

Adults: Take with food or milk. Urticaria: 20mg 2-4 hrs before UV exposure. Psoriasis: Initial: <30kg: 10mg, 30-50kg: 20mg, 51-65kg: 30mg, 66-80kg: 40mg, 81-90kg: 50mg, 91-115kg: 60mg, >115kg: 70mg. Take 2 hrs before UVA exposure. Titrate:





To turn off the highlight, press **BACK**.

2. Use the arrow keys to highlight a word.

Dosage and Administration

Adults: Take with food or milk. Urticaria: 20mg 2-4 hrs before UV exposure. **Psoriasis**: Initial: <30kg: 10mg, 30-50kg: 20mg, 51-65kg: 30mg, 66-80kg: 40mg, 81-90kg: 50mg, 91-115kg: 60mg, >115kg: 70mg. Take 2 hrs before UVA exposure. Titrate:

3. Press **ENTER** to search for it.

Hold  and press  repeatedly to see the Outline locations of the next matches, if any. To see the location of previous matches, if any, hold  and use .

4. Press **ENTER** to see the match.
5. Press **MENU** to see the menu items with matches.
6. Press **CLEAR** when finished.

Expanding Your Searches

You can expand a word search up to three times to find inflected forms of words, synonyms, and similar words.

1. Press **SEARCH**.
2. Type a word and then press **ENTER**. For example, enter *blood*.





Type words, press **ENTER**.

blood 396 Matches

Press **MENU** for other options.

3. Press **MORE** to expand the search.

Outline location:
ABBOKINASE OPEN-CATH R.
Indications and Usage
7 Matches

Hold  and use  to see the Outline locations of the next matches, if any. To see the location of previous matches, if any, hold  and use .

4. Press **ENTER** to see the match.
5. Press **MENU** to see the matches listed on the menu.
6. Press **CLEAR** when finished.

► Understanding Word Searches and Expansions

When you search for a word, either by entering it at the word search screen or by highlighting it in text, headings and subheadings (in bold type) are searched first for exact matches. If no exact matches are found in the headings, then the text is searched for exact matches.

If no exact matches are found, then the headings are searched for inexact matches, such as inflections and synonyms. If no inexact matches are found in the headings, then the text is searched for them.

A word search stops as soon as matches are found at a level described above.

Pressing **MORE** continues the search to the next level and finds additional matches, if any. **Note:** The number of matches found at a given search level may be less than at the previous level.

To Add a Note

You can add up to 10 notes to the text of this book to quickly find the sections that you most often use.

1. When text is on screen, press **LIST**.
2. Press **ENTER** to select *Add note*.


Type your note then press **ENTER**. Or press **HELP**.

EXCESSIVE

Press **STAR** **BACK** to cancel.

3. Do one of the following:
 - To accept the note title, press **ENTER**.
 - To change the note title, type your changes and then press **ENTER**.

Note titles can contain up to 24 characters.

- To cancel, hold  and press **BACK**.

NOTE **Indications and Usage**
Prevention of pregnancy.

Contraindications

Thrombophlebitis, DVT or thromboembolic disorders, pregnancy, cerebrovascular or coronary artery disease, undiagnosed abnormal

The **NOTE** marker is added to the text when you add a note.

Using Notes

To Find a Note

Once you have added notes to this book, you can easily find them.

1. **Press LIST.**
2. **Use ↵ to highlight a note title.**
Your notes are listed by recency.
3. **Press ENTER to go to that note.**

To Remove a Note

1. **Go to the text that contains the note you want to remove.**
To learn how, read “To Find a Note” above.
2. **Press LIST.**
3. **To see the full title of the note to be removed, press ?*.**
Press **BACK** to exit without removing the note.
4. **Press ENTER to select *Remove note*.**

► Removing All Your Notes


You can remove all your notes at one time by resetting the Medical Book System. To learn how, read “Resetting the Medical Book System.”

WARNING! Resetting the Medical Book System may erase other information that you have entered in other installed book cards.

Looking Up Words in Other Books

The Medical Book System platform enables you to look up a word from one installed book card in another installed book card.

This book card can send words to and receive words from other book cards. Some book cards are not able to transfer words. To learn if a book card can send or receive words, read its User’s Guide.

1. **Install two book cards in the platform.**
2. **Select *PDR 2004*.**
3. **Highlight a word.**
You can highlight words in menus, lists, entries, and other text. To learn how to highlight a word in this book, read “Highlighting Search Words.”
4. **Hold  and press CARD.**
5. **Highlight the icon of the other book card and then press ENTER.**
The word that you highlighted appears in the other book.
6. **Press ENTER again, if needed, to search for the word in the other book.**

Resetting the Medical Book System

If, due to electrostatic discharge or other cause, the keyboard fails to respond or the screen performs erratically, try the steps below to reset the platform. Perform only as many steps as required to restore normal operation.

CAUTION: Resetting the platform when book cards are installed may erase settings and information entered in those book cards. Remove all book cards before resetting.

1. Hold CLEAR and press ON/OFF.

If nothing happens, try Step 2.

2. Use the end of an opened paper clip to gently press the reset button on the back of the platform.

The reset button is recessed in a pin-sized hole located near the book card slots.

CAUTION: Pressing the reset button with more than light pressure may permanently disable it.

If nothing happens, try Step 3.

3. Remove and reinstall the batteries.

To learn how, read the User's Guide for the Medical Book System platform.


Book Card Care

- **Do not touch the metal contacts on the book cards.**

CAUTION: Touching the electrical contacts with statically charged objects, including your fingers, could erase information entered in a book card.



- **Do not put excessive pressure on the book cards.**

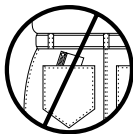

- **Do not expose the book cards to heat, cold, or liquids.**



Model PDR-3050

- size: 5.9 x 4.2 x 0.6 cm
- weight: 0.4 oz

© 2004 Franklin Electronic Publishers, Inc. Burlington, N.J. 08016-4907 U.S.A. All rights reserved.

© 2004 Thomson PDR. All rights reserved.

Registered trademark used herein under license.

Medical Book System and BOOKMAN are trademarks of Franklin Electronic Publishers, Inc.

FCC Tested to Comply With FCC Standards

FOR HOME OR OFFICE USE

U.S. Patents 4,830,618;

ISBN 1-59074-254-0

This unit may change operating modes due to Electro-static Discharge. Normal operation of this unit can be re-established by pressing the reset key, **ON/OFF**, or by removing/replacing batteries.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE: This unit was tested with shielded cables on the peripheral devices. Shielded cables must be used with the unit to insure compliance.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Limited Warranty (U.S. only)

LIMITED WARRANTY, DISCLAIMER OF WARRANTIES AND LIMITED REMEDY

(A) LIMITED WARRANTY. Franklin warrants to the original end user that for a period of one (1) year from the original date of purchase as evidenced by a copy of your receipt, your franklin product shall be free from defects in materials and workmanship. This limited warranty does not include damage due to acts of god, accident, misuse, abuse, negligence, modification, unsuitable environment or improper maintenance. The sole obligation and liability of franklin, and your exclusive remedy under this limited warranty, will be repair or replacement with the same or an equivalent product of the defective portion of the product, at the option of franklin if it determines that the product was defective and the defects arose within the duration of the limited warranty. This remedy is your exclusive remedy for breach of this warranty. This warranty gives you certain rights; you may also have other legislated rights that may vary from jurisdiction to jurisdiction.

(B) DISCLAIMER OF WARRANTIES AND LIMITATION OF LIABILITY. Except for the limited warranties expressly recited above, this franklin is provided on an "as is" basis, without any other warranties or conditions, express or implied, including, but not limited to, warranties of merchantable quality, merchantability or fitness for a particular purpose, or those arising by law, statute, usage of trade, or course of dealing. This warranty applies only to products manufactured by franklin and does not include batteries, corrosion of battery contacts or any other damage caused by batteries. Neither franklin nor our dealers or suppliers shall have any liability to you or any other person or entity for any indirect, incidental, special, or consequential damages whatsoever,

including, but not limited to, loss of revenue or profit, lost or damaged data or other commercial or economic loss, even if we have been advised of the possibility of such damages, or they are otherwise foreseeable. We are also not responsible for claims by a third party. Our maximum aggregate liability to you, and that of our dealers and suppliers, shall not exceed the amount paid by you for the franklin product as evidenced by your purchase receipt. You acknowledge that this is a reasonable allocation of risk. Some states/countries do not allow the exclusion or limitation of liability for consequential or incidental damages, so the above limitation may not apply to you. If the laws of the relevant jurisdiction do not permit full waiver of implied warranties, then the duration of implied warranties and conditions are limited to the duration of the express warranty granted herein.

(C) WARRANTY SERVICE: Upon discovering a defect, you must call franklin's customer service desk, 1-800-266-5626, to request a return merchandise authorization ("rma") number, before returning the product (transportation charges prepaid) to:

Franklin Electronic Publishers, Inc.
Attn: Service Department
One Franklin Plaza
Burlington, NJ 08016-4907

If you return a franklin product, please include a note with the rma, your name, address, telephone number, a brief description of the defect and a copy of your sales receipt as proof of your original date of purchase. You must also write the rma prominently on the package if you return the product, otherwise there may be a lengthy delay in the processing of your return. We strongly recommend using a trackable form of delivery to franklin for your return.

Index

- ?* key 4, 10, 11, 14
- Arrow keys 5, 7
- Automatic shutoff 6
- Blue (LIST) key 4, 13
- Book cards
 - installing 6
 - platform compatibility 2
 - removing 6
 - selecting 6
- Changing
 - screen contrast 7
 - shutoff time 7
 - type size 7
- Color keys
 - key guide 4
 - understanding 5
- Default state
 - changing 7
 - defined 7
- Demonstration 7
- Direction keys 4
- Equivalents, finding 9
- Function keys 4
- Green (SPEC) key 4, 10
- Help messages 7
- Highlighting words 12
- Key combinations 4
- LIST 4, 13
- Misspellings, correcting
 - drug names 9
 - search words 11
- MORE key 4, 12
- NOTE marker 13
- Notes 13
 - deleting 14
 - deleting all at once 14
 - finding 14
- Pocket PDR™
 - description of 3
- Quick Keys 5, 9, 10
 - understanding 5
- Red (MORE) key 4, 12
- Reset button 15
- Screen contrast 7
- SEARCH key 4, 10, 12
- Shutoff time 7
- SPEC key 4, 10
- Therapeutic classes, finding 9
- Tutorial 7
- Type size 7
- Viewing
 - demonstration 7
 - help messages 7
- Word searches, understanding 13
- Words, highlighting 12
- Yellow (SEARCH) key 4, 10, 12



Medical Book System[™]